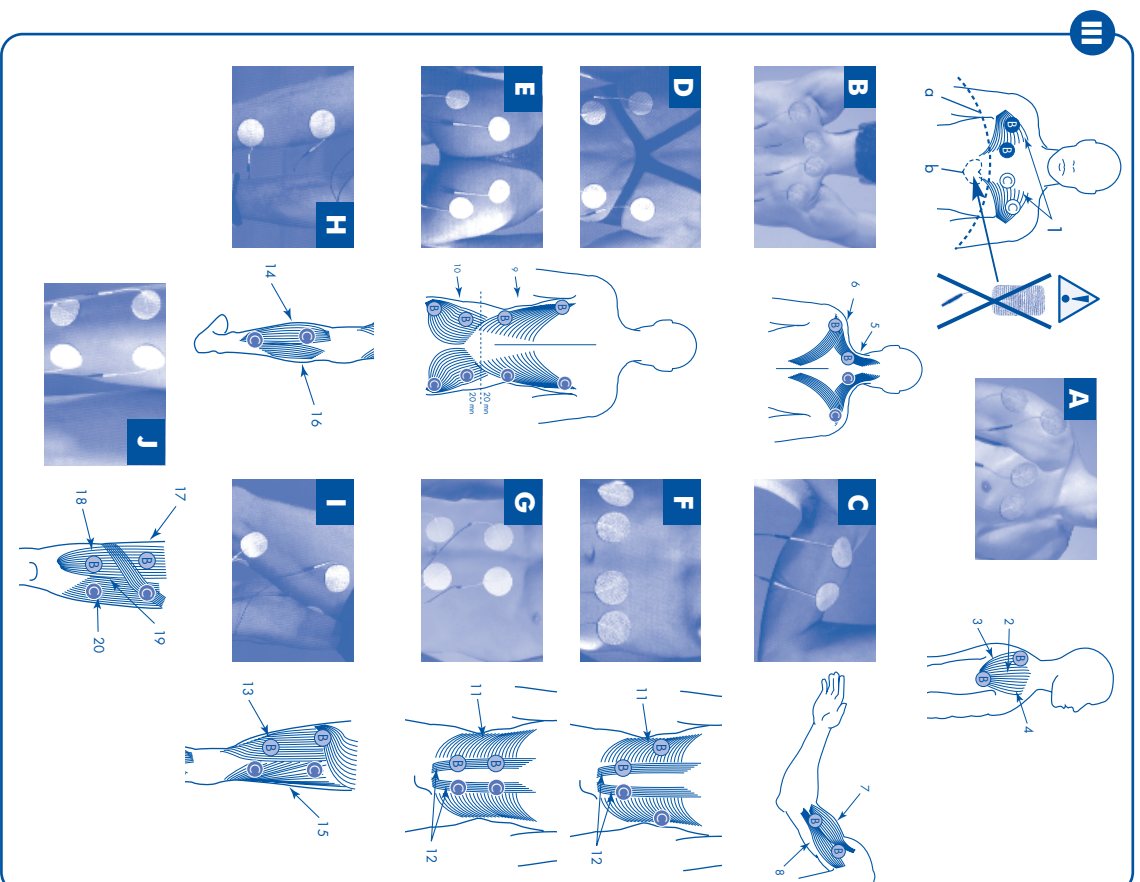
GÉNÉRALITÉS

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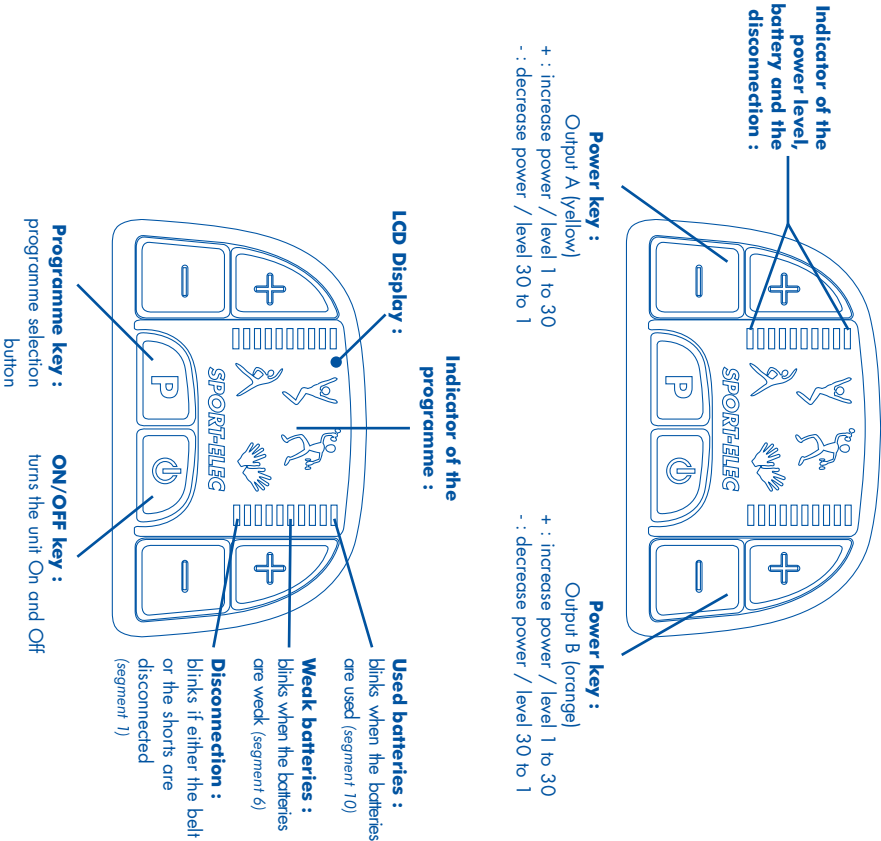
Description of the unit
BODY CONTROL 4

NB : the segments 1, 6 et 10 have a double function.

Programs

PG 01 **Well-being**

20 minutes to warm up and to tone your muscles.
Total duration : 20 min

PG 02 **Energie**

40 minutes of body building to firm up, but also to increase muscular volume after a period of inactivity.
Total duration : 40 min

PG 03 **Fitness**

20 minutes of more intensive musculation to work the target areas.
Total duration : 20 min

PG 04 **Relaxation**

20 minutes to soothe tired muscles and allow you to feel the well being.
Total duration : 20 min

**You have just acquired an effective and powerful professional unit.
Take your time to discover how to use it.**

BEFORE USING THIS UNIT

- 1 - Read the instructions carefully
- 2 - Before using, insert batteries in your apparatus (see photo II - A page 1)
- 3 - Test your unit on your biceps using the quick test procedure below

OPERATION QUICK TEST

TESTING YOUR UNIT ON YOUR BICEPS

- 1 - Connect 2 adhesive electrodes to your device (see procedure of connection in the instructions for use).
- 2 - Test the device with the program PG02.
- 3 - Increase power on this channel gradually.
- 4 - You would begin to feel contractions, the device is working.
- 5 - This discovery procedure shows your capacity in using the unit's level of performance.

CUSTOMER SUPPORT + 33 (0)2 32 96 50 50

GENERAL

SPORT-ELEC Body Control 4 is conceived for women and men who try to be fit. **SPORT-ELEC Body Control 4**, is a Be slim device which uses electrical stimulation through the "contraction-release" effect on muscles that you want to work.
The aim of the **SPORT-ELEC Body Control 4** is the following: to refine your silhouette by acting on the muscular masses (waist, Abs, butt muscles, thighs, and legs). **SPORT-ELEC Body Control 4** contains 4 programmes for a more particular development of certain muscles (pectoral, abdominal, biceps...) resulting from **SPORT-ELEC** research. The frequencies and impulses width of each programme have been defined to meet perfectly everybody's requirements :

- Work the muscle gently and facilitate its recovery after an effort
- Improve the firmness of tissues
- Reshape the silhouette

In conclusion, according to our activity, some muscles are more or less developed, the others atrophy on the contrary. Our body contains more than 600 muscles. The shape and harmony of our body depends on these muscles. **SPORT-ELEC Body Control 4** allows activating the set of muscles without any effort from you. Its daily use will restore vigour of your muscle structure.

I. INSERTION OF BATTERIES

Check that the apparatus is switched off. Open the "batteries" compartment, insert three 1,5 V AA LR6 batteries into the compartment then to close the case. Comply with + and - indications (photo II - A page 1). For an optimal use of your apparatus, it is advised to use good quality batteries.

II. STARTING THE UNIT

A / PRESENTATION OF THE UNIT

Your electrostimulation device is an appliance in the point of the technique, it respects the current standards in the field of medical application. The interface "man -machine" is simplified and redized by means of pictograms and function keys. One of four, the pictogram in black indicates you the program chosen by the "P" key.
Two scales (1 by output) indicate you 30 power levels.

- You also find 6 order keys (scheme, page 3) :
- 1 ON/OFF (red key O) - a short press
 - 1 "P" (grey key) to select the programs - a short press
 - 2 "+" / "-" yellow keys and 2 "++" / "--" orange keys to adjust the intensity of the stimulation (a short press).
- A long (1 sec.) press on the "u" key allows to return the power level rapidly to zero.

B / UNIT FUNCTION

STEP 1 - Disconnection

Your device is endowed with an accessories disconnection detector.
If there is no contact with the human body and the accessory, the power is stopped on the concerned output (channel A = yellow / channel B = orange).
You are warned of a disconnection by blinking of a segment during 30 sec., after 30 sec. the blinking stops (diagram, page 3).
If no accessory is in touch with the body, the power cannot be emitted.

STEP 2 - Batteries

- If the batteries are "good", there is no blinking.
- If the batteries are "weak", two segments (the 6th segment) blink (**scheme page 3**). Your device still works.
- If the batteries are "used", two segments (the 10th segment) blink (**scheme page 3**). Your device stops, even in the course of a program. Change the batteries.

STEP 3 - ON / OFF Switch "O"

- A short press on the "O" key puts the device under or off power.
- Every time under power, the device makes an auto-test on 11 essential parameters at the level of the safety of its operation. During the auto-test segments and pictograms are lit during 0,5 second.
- If an internal defect in the device is detected, all the pictograms flash. Send back the device to the After Sale Service.
- The auto-test does not take into consideration the used batteries. When the batteries are used the auto-test cannot be performed.
- One long press (over 3 sec) on the "O" key sits up the auto-test. In this case, stop the device by pressing the "O" key and then press shortly the "O" key again.

STEP 4 - Choice of a program "P"

It is enough to make one or several presses on the "P" key, the programs ravel. The programs are announced by pictograms. As soon as a program is selected, corresponding pictogram lights (**see page 3**). Once the program is chosen, you can start it by increasing the power on the only of power keys "+". During the program the key "P" is inactive.

STEP 5 - End of program - Stop

- At the end of the session the device stops automatically.
- To stop the device in the course of program, you just have to turn 2 power keys to zero or to press the "O" key.
- In case of emergency, press the "O" key.

STEP 6 - Adjustment the power

You have 30 levels from 0 to 30 to adjust the power (there are three power levels between two segments).

III. GENERAL INSTRUCTIONS

STEP 1 : Take the electrodes and remove carefully their protection, dampen them one by one with water on your fingertips on the adhesive gel, and wait for 2 minutes (**photo I - B / C page 1**).

STEP 2 - Electrodes function

Fit the electrodes on your body as shown in diagram **III page 2** :
Positions are shown for information only.

Examples of use : A = connector channel A (yellow), C = connector channel C (orange)

A - Pectoral muscles	5 - Upper trapezius	H/I - Legs
1 - Major pectoral	6 - Middle trapezius	13 - Curral biceps
a - Do not place electrodes in the heart region	7 - Biceps longitudinal	14 - Half tendinous
b - Heart	8 - Biceps transversal	15 - Outer genellus
A - Shoulders	D/E - Back and buttocks	16 - Inner genellus
2 - Middle deltoid	9 - Main back muscles	J - Thighs
3 - Front	10 - Main gluteal muscles	17 - Major adductants
4 - Back	F/G - Abdominal muscles	18 - Vastus internus
B - Trapezil	11 - Major oblique	19 - Rear longitudinal
	12 - Major longitudinal	20 - Vastus externus

Plug the electrodes into the connectors of the device (**photo I - D page 1**). The Body Control 4 consists of 2 outputs of two 90cm long wires ending in 2 groups of 2 connecting plugs (2 yellow and 2 orange plugs). You may attach your **SPORT-ELEC®** to the inside of your clothes or underwear with its clip.

STEP 3 : Start your device by following the procedure in "**Starting the unit**" detailed previously in paragraph II.

The contractions must be firm but never painful. If you feel an unpleasant sensation around electrodes : decrease the power to minimum, move the electrodes and gradually increase the power. The adjustments being optimized in this way, let your device work during the full duration of the programme.

STEP 4 : Once the programme has ended, unplug the electrodes of connectors. Gently remove the electrodes, moisten them slightly with a few drops of water and put them carefully back to their protection holder. Store the electrodes in a plastic bag in a cool place (vegetable compartment of a fridge).

CONTRA-INDICATIONS
Do not use SPORT-ELEC® :

- * on the heart area
- * If you carry a pacemaker
- * If you are pregnant
- * during first 6 weeks after baby birth
- * In case of neuromuscular diseases
- * In case of hemorrhagic diseases

IV. Recommendation of use

The unit is recommended for physiotherapists for functional re-education.

WARNINGS :

Do not use the device by driving or by working on a machine.

Do not use this electro stimulation unit in a damp environment, a bathroom, a sauna, or close to electric machinery connected to mains and earthed, or even piping earthed.

The simultaneous connection of a patient to a high frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes, and the stimulator may be damaged. Operation in close proximity of a short wave therapy unit may cause instability in the stimulator's output power.

RECOMMENDATIONS :

Please apply regulations enforced in your country when disposing your product at its end of life.

Therefore, please discard it in a location planned to this effect, ensuring it is disposed in an environment-friendly manner

PRECAUTIONS OF USE

- * Do not use the unit on your face
- * Do not leave within reach of children
- * **Ask your doctor's advice :**
- if you have electronic or metallic implants (pins, etc.)
- in case of skin condition (wounds,...)

- **Use only on those parts of the body for which the accessories were designed.**
- **Functions only on batteries**
- **Not to connect to another source of power**

RECOMMENDATIONS OF USE

- You can restart a program's cycle on another muscle zone or change the program while remaining on the same muscle zone, without exceeding however 40 min use per muscle zone and per day.
 - For optimum use, refer to the diagrams shown in this leaflet.
 - Clean the unit and its cables with a cloth dampened with a disinfecting solution (alcohol type) diluted in water at 1/10th.
 - The adhesive electrodes have a lifetime of about 40 uses, you can order them directly from the company headquarters or from your reseller
- SPORT-ELEC Institut - BP335 - 31 Rue du Val Breton - 27520 BOURGHEROULDE - FRANCE**
Tél. 02 32 96 50 50 - Fax. 02 32 96 50 59 - www.sport-elec.com / info@sport-elec.com

SAFETY CHECK

Every 24 months, check the following points :

- 1 - Check for superficial degradation of the equipment
- 2 - Check proper operation of the unit :
 - * legible display
 - * switch, contact
- 3 - Measure electric values :
 - * with a 500-ohm load, the current must not exceed 100 μ A.
- 4 - Check reliability of accessories :
 - * adhesive electrodes in good condition (no tears, no material defects)
 - * wire of adhesive electrodes in good condition.

STANDARDS

(Directive 89/336/CEE)

Unit : EN60601-1 EN60601-2-10

Hydrophilic and hypodallergenic adhesive electrodes ($\varnothing$ 50 mm) : 93/42/CEE

Classification type BF

V. TECHNICAL SPECIFICATIONS

UNIT :	
Type of current	Dual phase
Number of programmes	4 programmes
Number of phases variations	79
Number of adjustable independent outputs	2
Current	From 0 to 70 mA - From 0 to 1000 Ω
Frequency range	1 to 100 Hz
Pulse width range	160 to 320 μ s
Power	3 x 1,5 V batteries type LR6 (not provided)
Unit dimensions and weight	69 x 43 x 87 (h) mm - 106 g
Automatic stop at end of each program	Yes - about 4 min 30 sec
Activation safety	100%
Detection absence of electrodes or of contact	yes detection independent on each of the 2 outputs
Indicator of power	bar graph - LCD display
Indicator of weak and/or defective batteries	yes - LCD display
Indicator absence contact electrodes	yes - LCD display
Indicator of the programs	pie diagrams - LCD display
Temperature of storage	0°C to + 45°C
Relative humidity of storage	10% to 90%
Temperature of use	+ 5°C to + 45°C
Relative humidity of use	20% to 65%
Connection device / electrodes	wires - plugs $\varnothing$ 2 mm mole
Caution : Read the manuel thoroughly	
RoHS : The unit are free of materials likely to have a negative impact on the environment	RoHS (2002/95/EC)
Guarantee	2 years

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